

Art.-Nr. ANOKRYO™-Combi-Set PZN 3941536  
Art.-Nr. ANOKRYO™-Gel PZN 7470157

# ANOKRYO®

*...cooling the  
pain down!*



**PakuMed**  
medical products gmbh

Im Löwental 79 · 45239 Essen | Germany  
Phone: +49 201-43 70 97-0 · Fax: +49 201-43 70 97-29  
E-Mail: info@pakumed.de · Internet: www.pakumed.de

# ANOKRYO®

Basic help in treating proctologic disorders  
low temperature rod and dilatator




For the treatment of:

- hemorrhoids 1° and 2°
- **acute anal fissure**
- perianal thrombosis
- anal discomfort
  - post-operative pain
  - itching
  - in pregnancy and lactation
  - at cryptitis, papillitis and after irradiation
  - and associated constipation

## Usage instruction

The rod reaches its lowest temperature of minus (-15°C) after approx. 1 – 2 hours of storage in the freezer (-15° – -20°C). The rod is then lubricated with a water soluble lubricant and is subsequently inserted into the anal canal for 5 – 10 Min. After cleaning and disinfection the rod is once again stored in its case in the freezer. ANOKRYO® can be used repeatedly and contains no harmful contents, is odorless, hygienic and causes no staining.

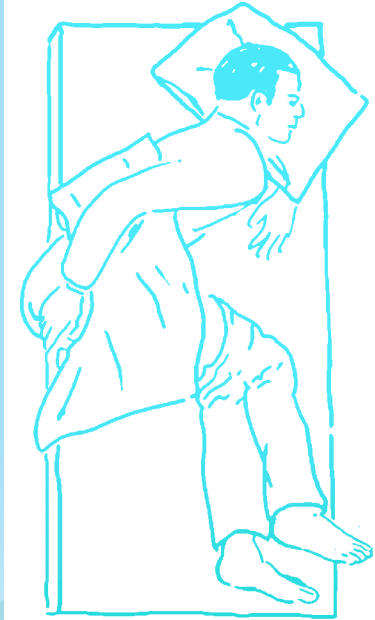

## Routine modes of treatment

hemorrhoid	anal fissure	perianal thrombosis and thrombosed hemorrhoidal prolapse
01. local preparations, ointment	01. local preparations laxatives	01. conservative treatment
02. sclerosing injections	02. infiltrations with local anesthetics, application of adstringents such as silver nitrate solution	02. operative treatment
03. rubber band ligation	03. quinine solutions	a) incision and thrombectomy
04. infrared coagulation	04. anal dilatation	b) excision
05. laser treatment	05. sphincterotomy with or without excision of the fissure	c) hemorrhoidectomy
06. cryotherapy	06. application of nitroglycerine	d) dilatation
07. anal dilation	07. botulin toxin	
08. spincterotomy		
09. hemorrhoidectomy		
10. stapler-hemorrhoidectomy		
11. ligation of the A. hemorrhoidalis		

Those treating of these conditions are familiar with advantages and disadvantages of the above mentioned modes of therapy. Often a single procedure will not suffice to achieve healing. Most conditions do indeed have a tendency to heal spontaneously, this may however require a considerable amount of time, during with patients my have severe complaints.

Although the etiology of these conditions is not always clear, it appears that a permanent elevation of anal sphincter muscle tone is common to all, suggestion a contributing role in their development.

ANOKRYO® by combining low temperature with dilatation, offers an effective mode of treatment to safely and rapidly alleviate the patient's symptoms and accelerate healing.

This is achieved by cold application at temperatures of minus 15°C and dilatation to 15 mm with the ANOKRYO® rod. The duration of a single treatment is 5 – 10 minutes and should commence 2 – 3 times daily. Venous return is promoted and arterial inflow is reduced thus reducing sphincter muscle tone.

[www.anokryo.de](http://www.anokryo.de)

This is to turn relieves pain and inhibits inflammation.

The use of low temperature application and dilatation in treating anal conditions is gaining popularity because it is effective, there are no untoward effects and is most suitable for home use.